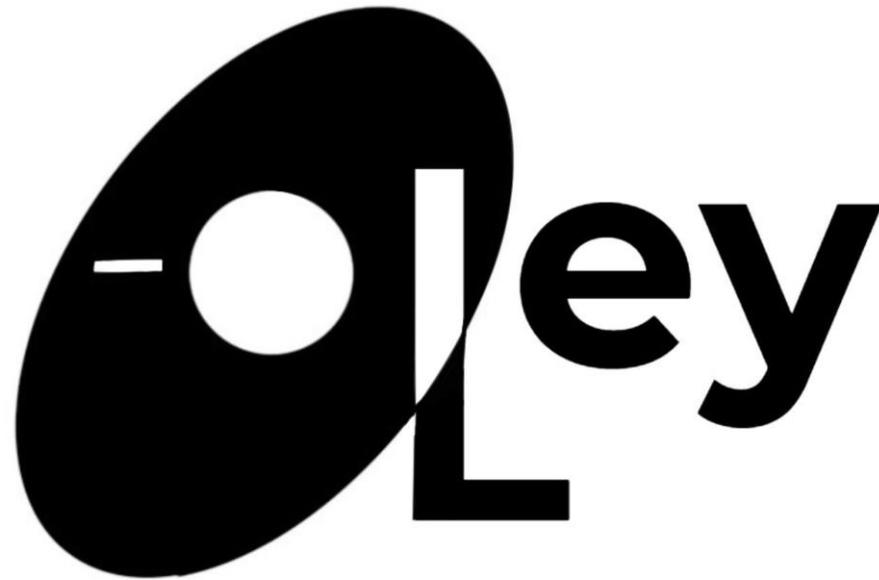
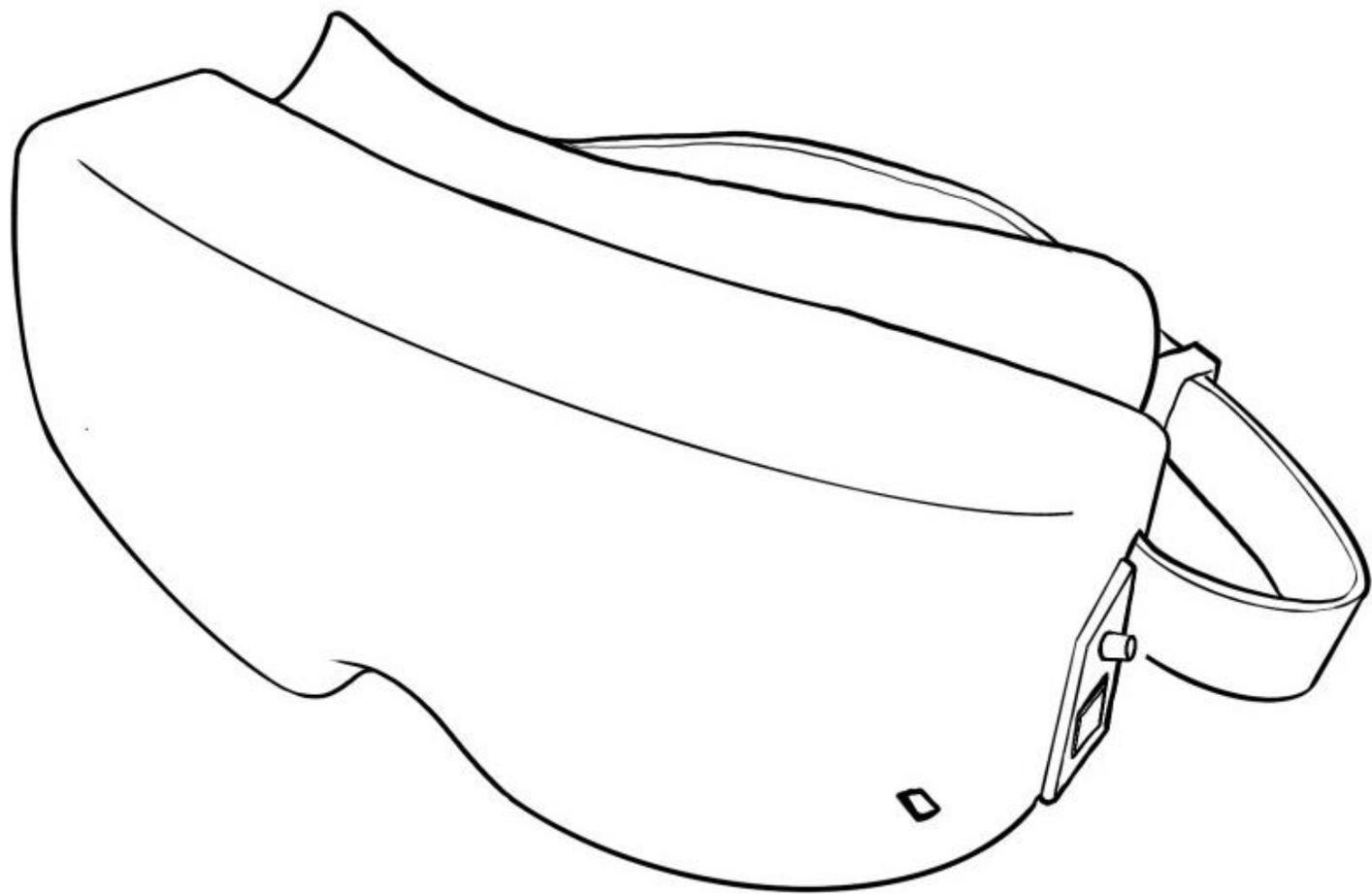
OOley Care Inc. Limited

User Manual

Product Overview

O-Oley

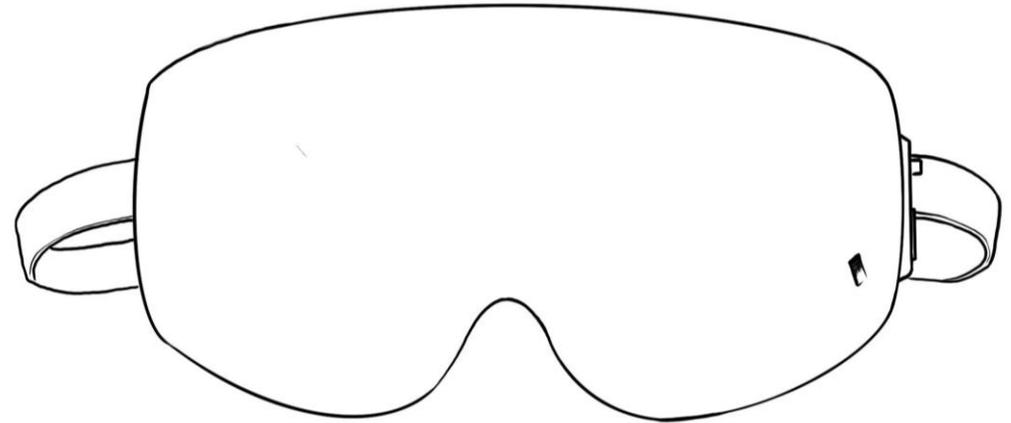
If I care more, eyes stress less

Created by the Hong Kong James Dyson Award National winners comes O-Oley, a wearable, ocular biomedical device that stops the progression of glaucoma. Using personalised thermal and pressure profiles, O-Oley aims to keep the eyes healthy by reducing ocular stress and improving ocular compliance. Enjoy the therapeutic effects to relax, relieve eye strain and improve quality of sleep while ensuring the long-term effects of healthier eyes.


Package Contents

- A O-Oley goggle
- B USB-C cable
- C User Manual


A

B

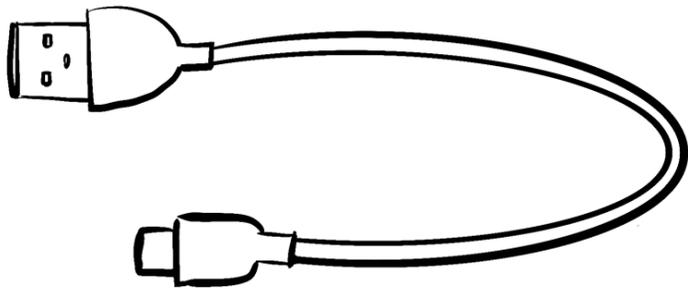C

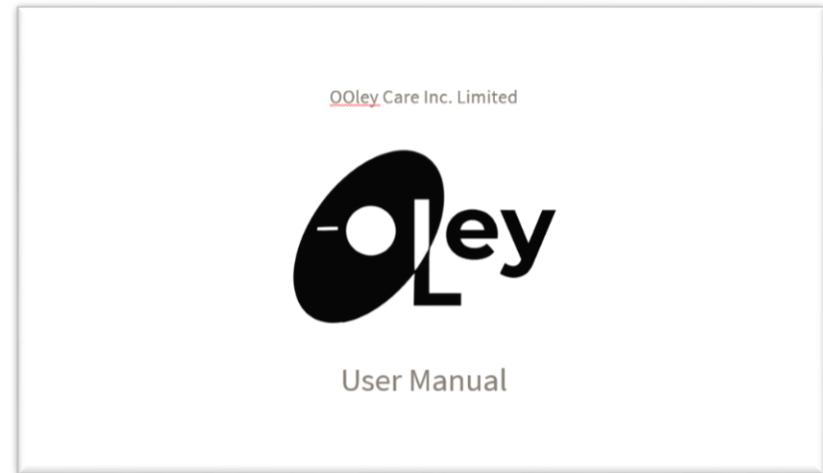User Interface

Power Button

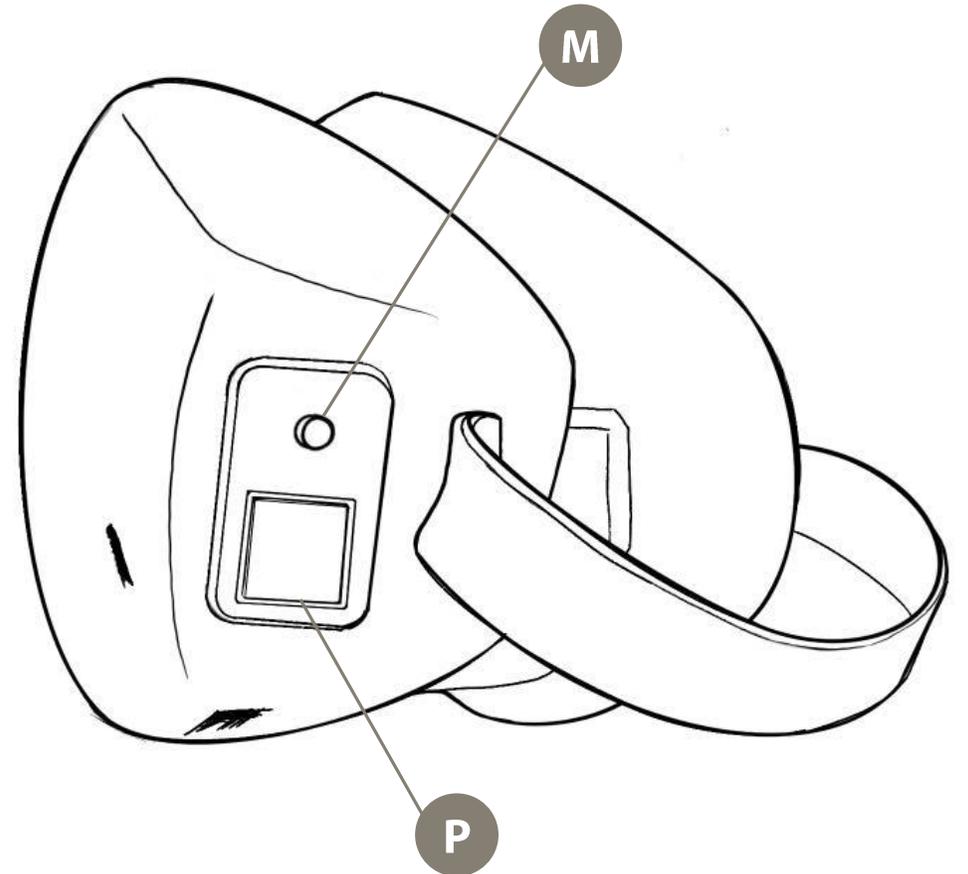
Press the power button once until a blue light indicator appears to power on.

Relaxation Mode Button

Press the relaxation mode button once to power on and off the therapeutic effects.


To power off, press the relaxation mode button once and the power button twice.


User Interface

Red Light

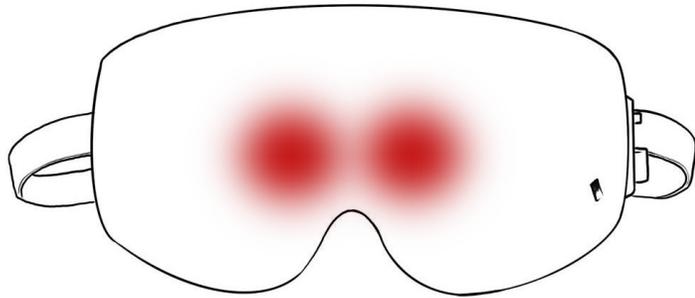After turning on the relaxation mode, the red light will appear to indicate the goggle charging up to start the therapeutic treatment.


Cyclic Therapy


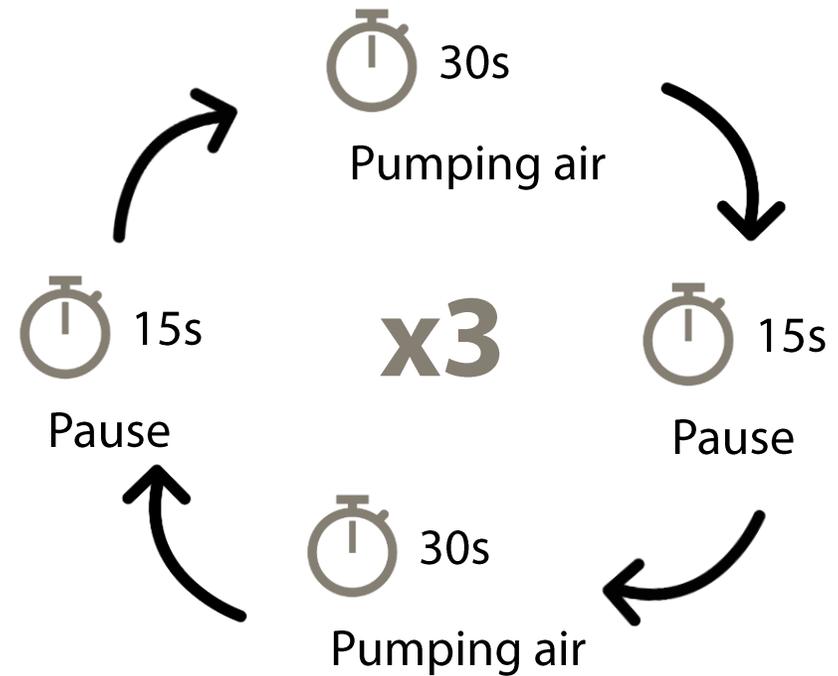
Getting Started

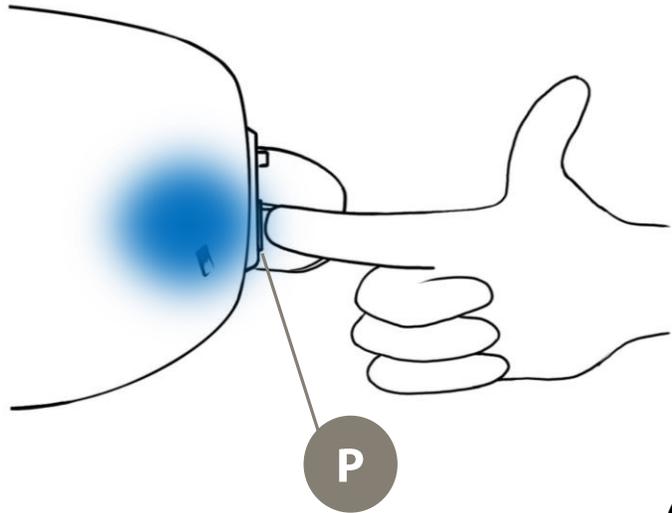1. Turn on the goggle by pressing on the power button **P** once until a blue light indicator is on.

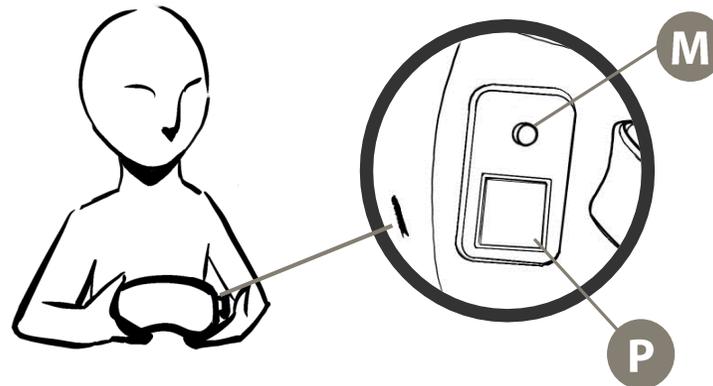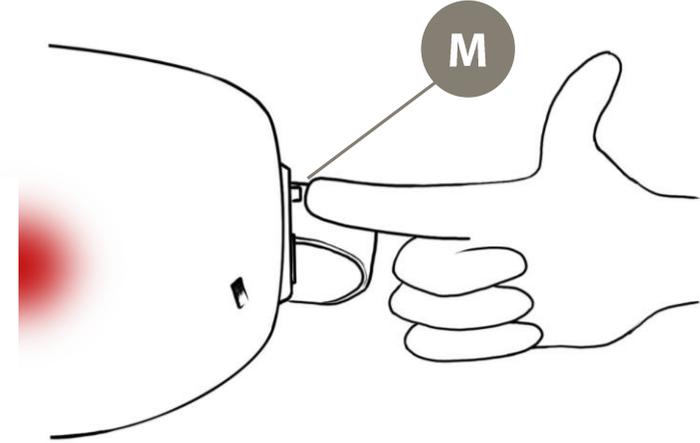
2. Start the relaxation mode by pressing on the white button **M**. The goggle will illuminate a red light in the middle


Getting Started

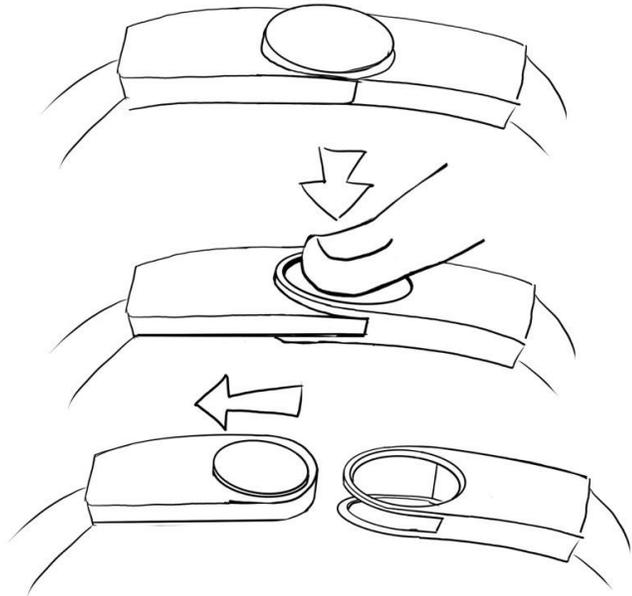3. Open the straps by pressing and pulling the lock out

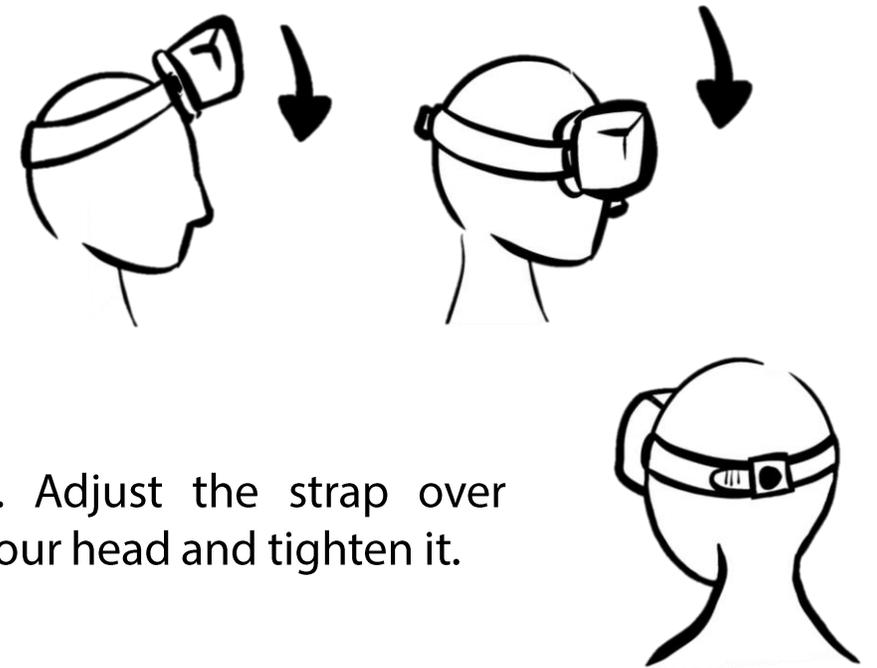4. Place the device on clean, dry skin and over the eyes. Ensure that eyeglasses and contact lenses are removed before use.

5. Adjust the strap over your head and tighten it.

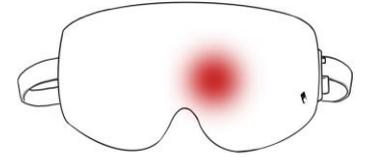
Getting Started

6. Enjoy the therapeutic effects on the eyes.


1

1 min – Pre-heating


2

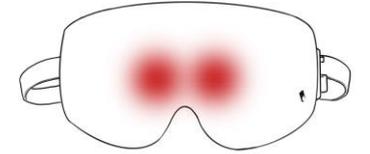2-3 mins – Thermal Stretching x3


3

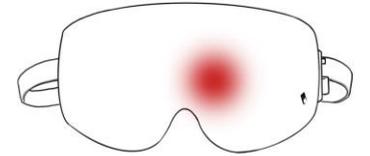10 mins – Continuous heating


4

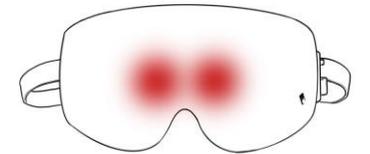2-3 mins – Thermal Stretching x3




5

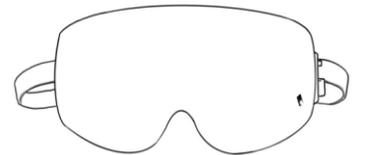Treatment completed


Getting Started

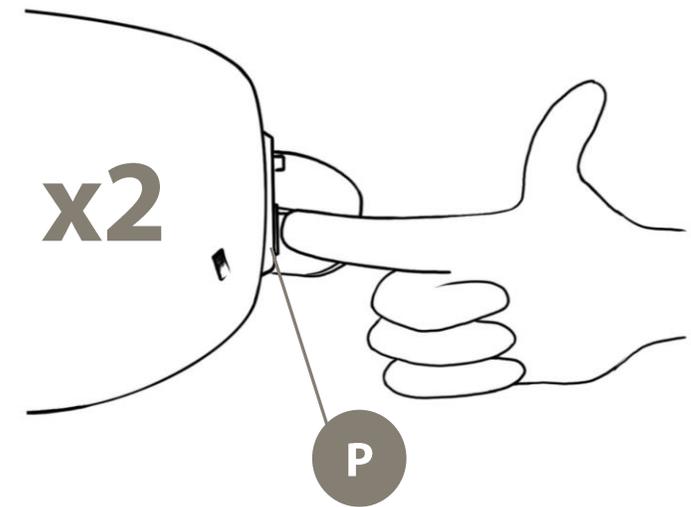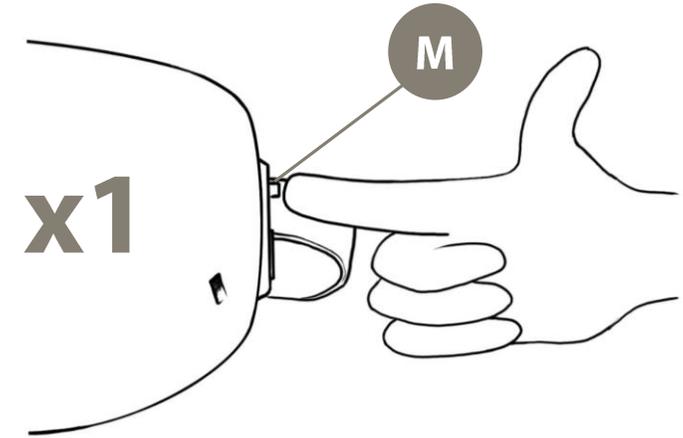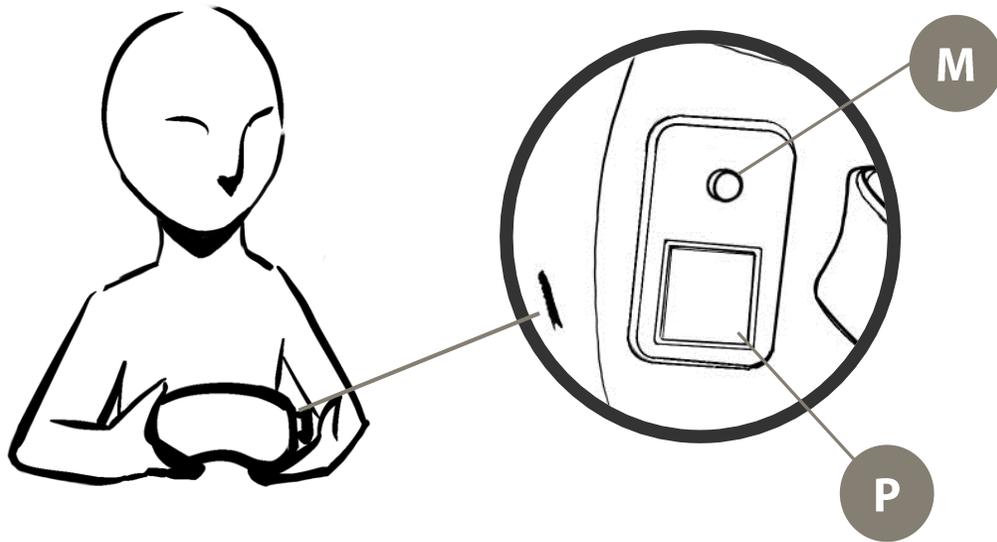7. Once you are done with your eye therapy session, turn off the device by pressing the relaxation mode button **M** once followed by the power button **P** twice.

Charging

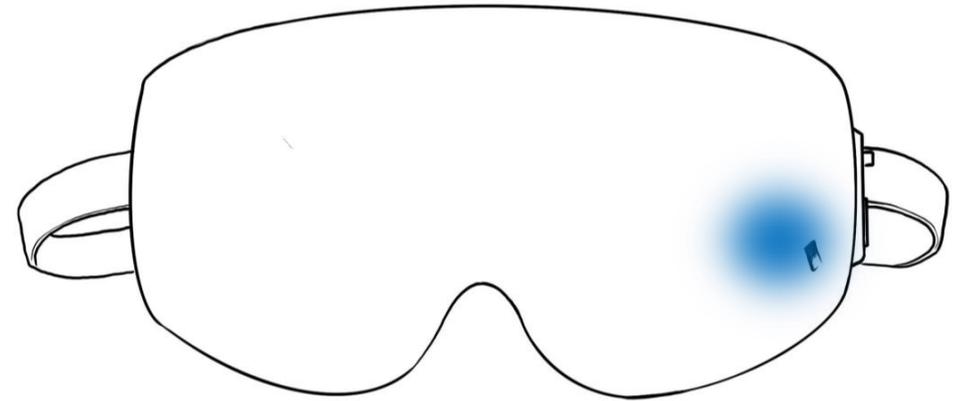- O-Oley is USB-C enabled
- The USB port is located at the bottom side of the device near the power button
- Insert the USB-C cable into the port to charge O-Oley
- A blue blinking light indicates the device is being charged
- The device is fully charged when the blue light is no longer blinking
- The device will take 30 minutes to be fully charged
- Charge the device after every session

Precautions

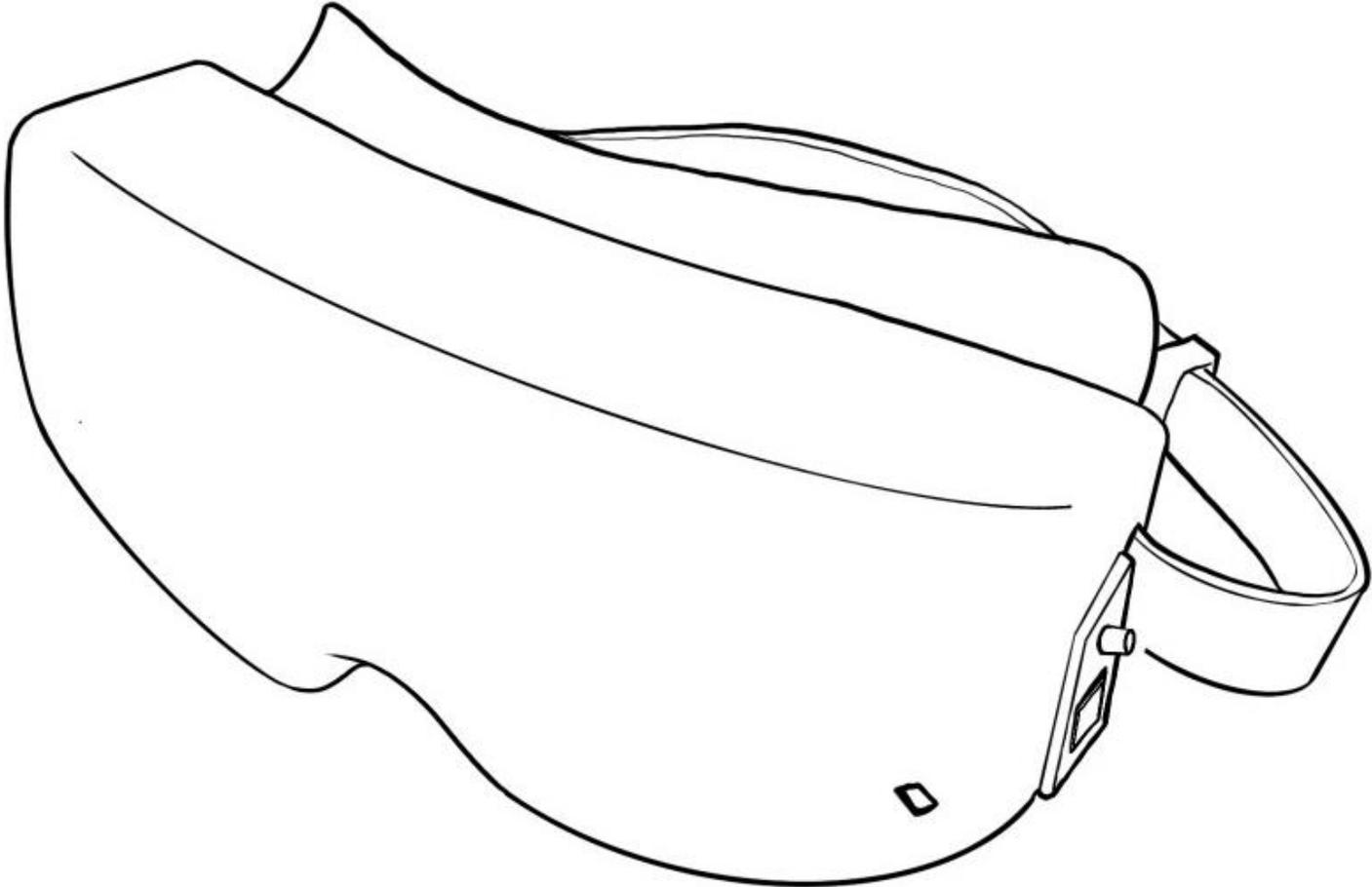
Warning

- O-Oley goggle is not waterproof and not water-resistant
- Do not open the case/shell of the device

Cleaning the goggle

- Wipe with clean soft dry cloth
- Use a diluted neutral detergent to clean

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