


O-Goggle Pro



**Eyecare device
to reduce glaucoma risk
with eye hot-yoga therapy**

This package includes O-Goggle Pro, a USB-C cable, and a warranty card.

User Interface

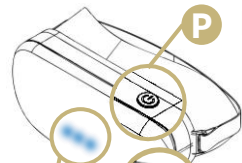


P Power button

Press the button **once** to power **on/off**


C USB-C port

Plug the USB-C cable to charge


L Light indicators

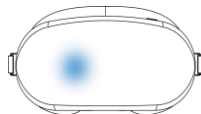

Power is on

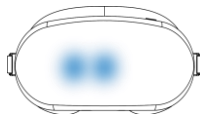

Heating starts

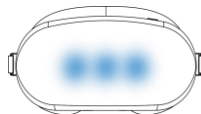

Hot Yoga starts


Getting started

Before the therapy



- Charge the device fully before use
- Remove eye glasses and contact lens
- Place the device on clean, dry eyes

During the 15-minute therapy

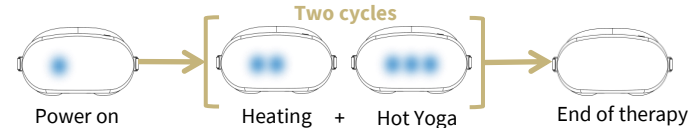

Precautions

- O-Goggle is **NOT** waterproof or water-resistant.
- Do **NOT** open the case/shell of the device
- Wipe with clean soft dry cloth to clean the device

O-Goggle Pro




使用者介面



護眼裝置
以眼部熱瑜伽療法
降低青光眼風險

內容物：O-Goggle Pro、USB-C 線、保證書。

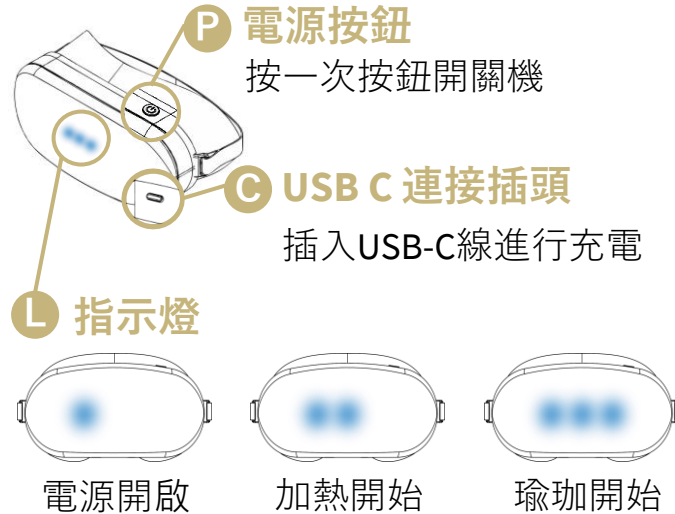


使用步驟

使用前準備



- 使用前請將裝置充飽電
- 取下眼鏡及隱形眼鏡
- 將設備輕貼於清潔的眼周肌膚

15分鐘療程期間

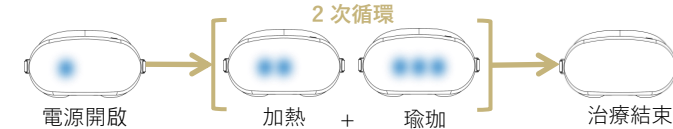

注意事項

- O-Goggle 不防水
- 請勿拆解裝置外殼
- 清潔時請使用乾淨柔軟的乾布