


OOley Care Inc. Limited



User Manual

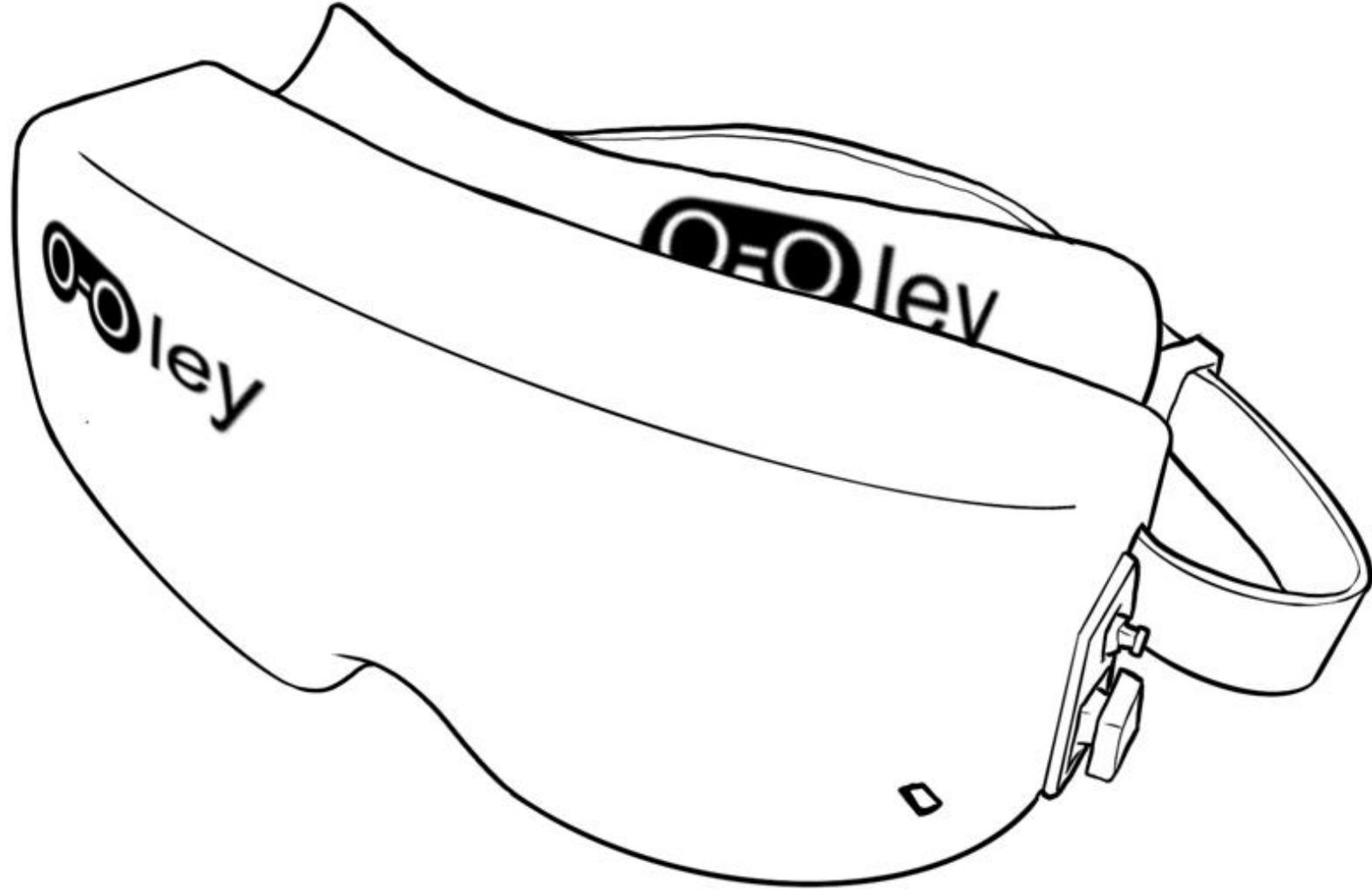

Product Overview




O-Oley

If I care more, eyes stress less

Created by the Hong Kong James Dyson Award National winners comes O-Oley, a wearable, ocular biomedical device that stops the progression of glaucoma. Using personalised thermal and pressure profiles, O-Oley aims to keep the eyes healthy by reducing ocular stress and improving ocular compliance. Enjoy the therapeutic effects to relax, relieve eye strain and improve quality of sleep while ensuring the long-term effects of healthier eyes.

Package Contents

- A O-Oley goggle
- B USB-C cable
- C User Manual

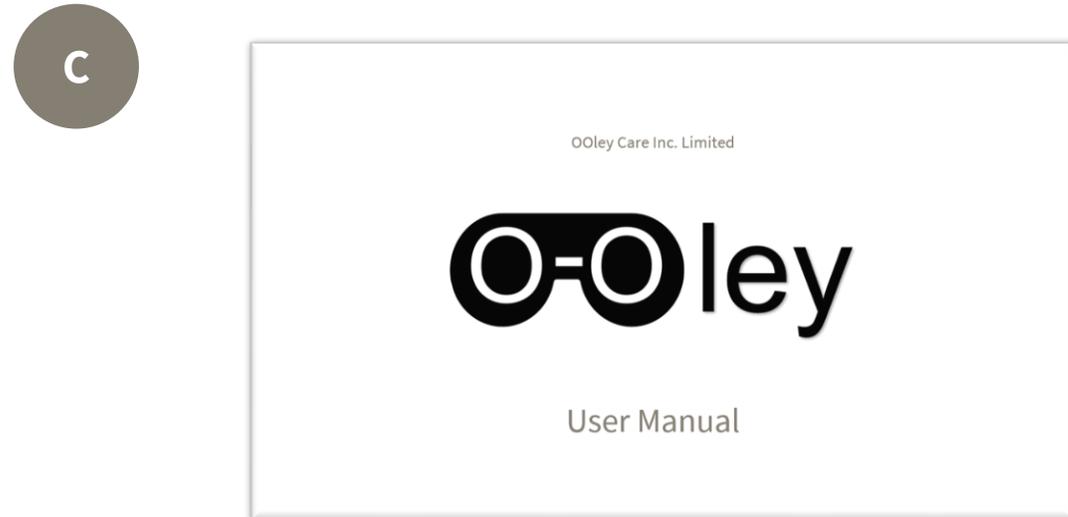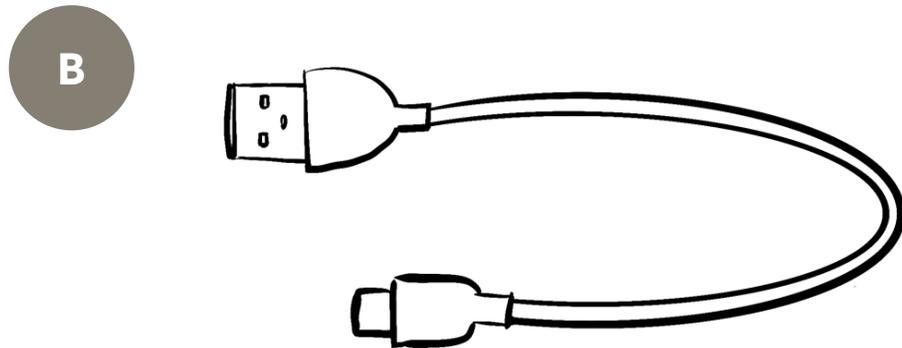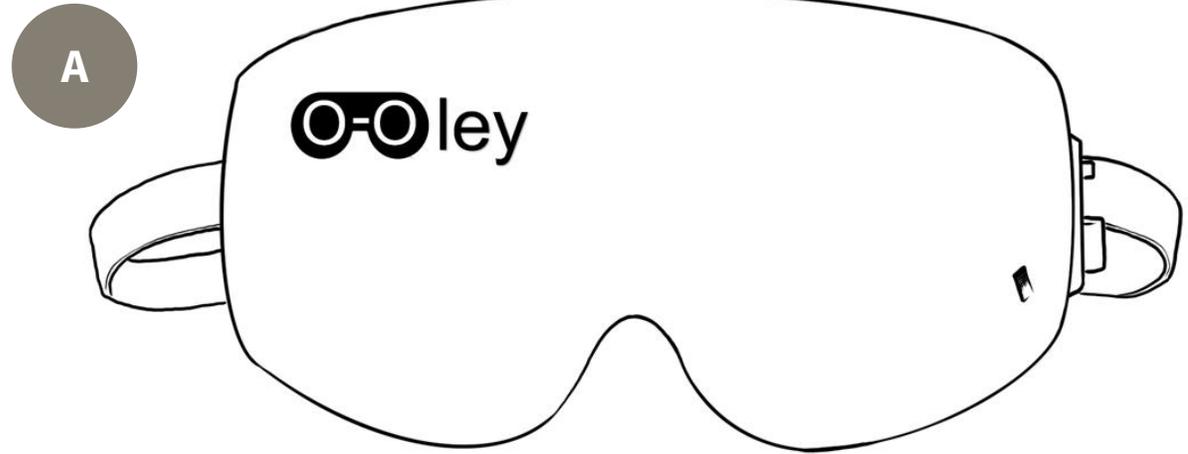

User Interface

P

Power Button


Press the power button once until a blue light indicator appears to power on.

Press the relaxation mode button once followed by the power button twice to power off.

M

Relaxation Mode Button

Press the relaxation mode button once to power on and off the therapeutic effects.

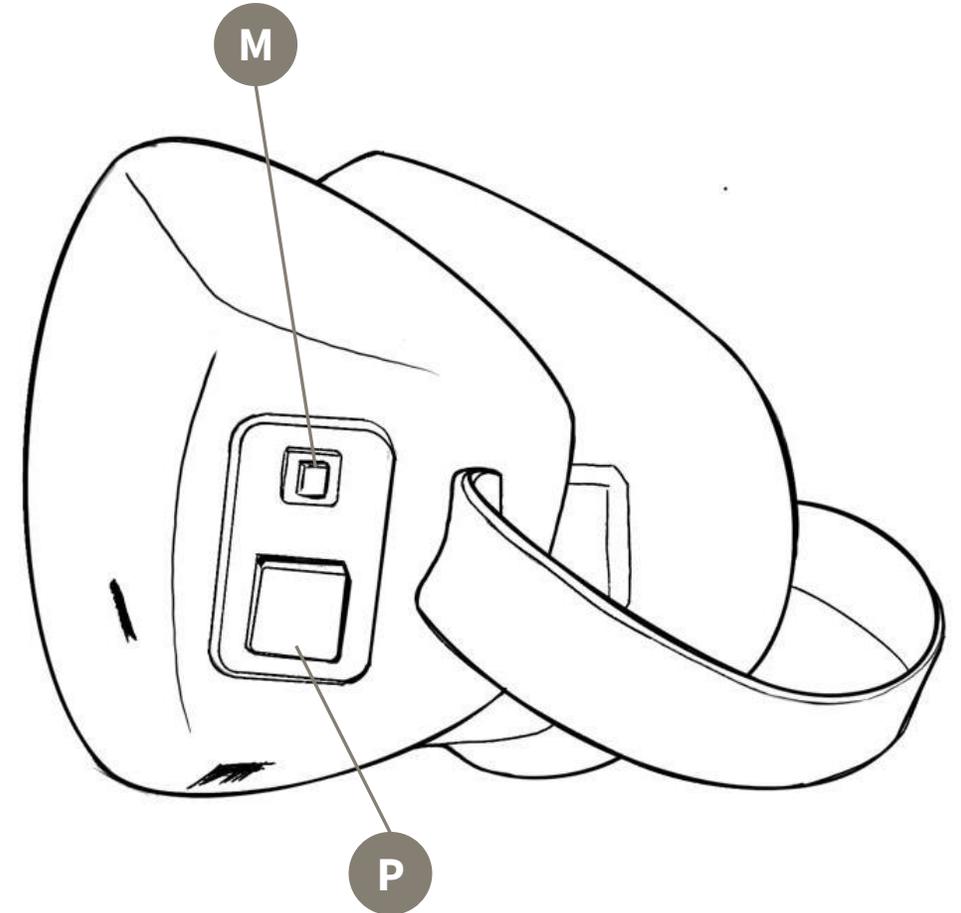

Light Indications

1

Red Light

After turning on the relaxation mode, the red light will appear to indicate the goggle charging up to start the therapeutic treatment.

2




Blue Light

After 3 seconds, the red light will turn to a blue light.

3

Green Light


After 2 seconds, the blue light will turn to green. This indicates that the goggle is ready to be worn.

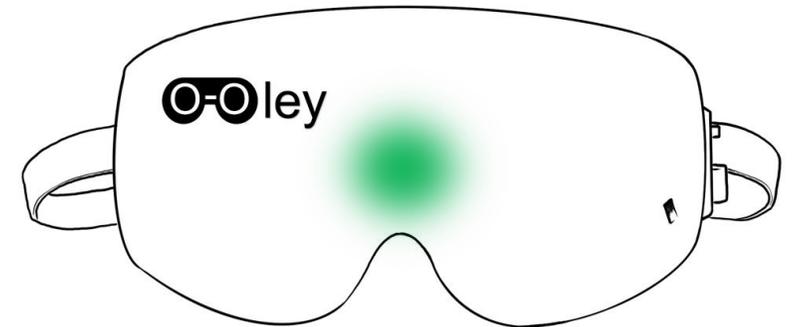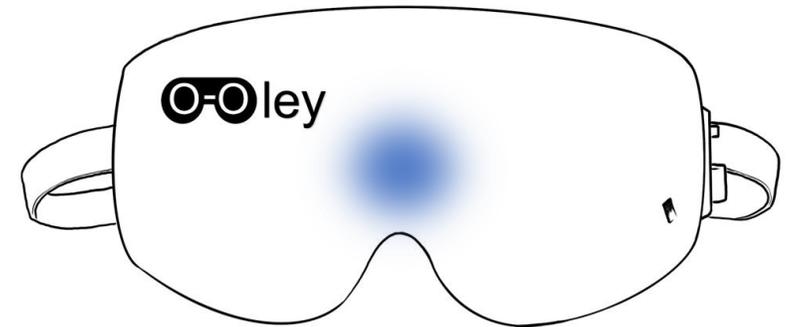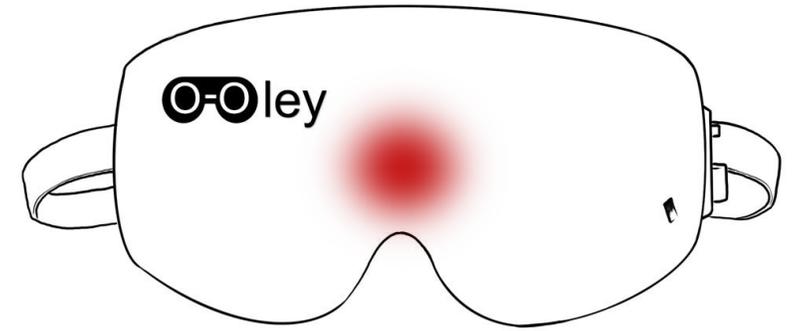


Getting Started

1. Turn on the goggle by pressing on the power button **P** once until a blue light indicator is on.

2. Start the relaxation mode by pressing on the white button **M**. The goggle will illuminate a red light.

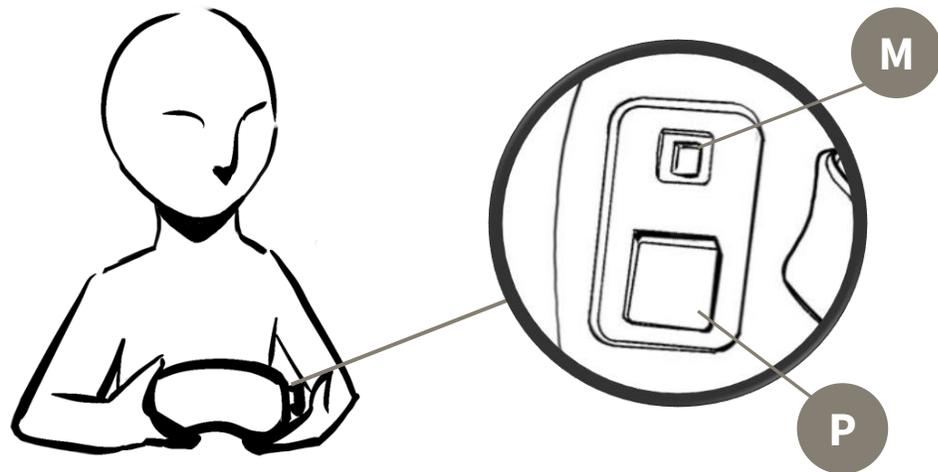


3. Wait until the red light becomes green.

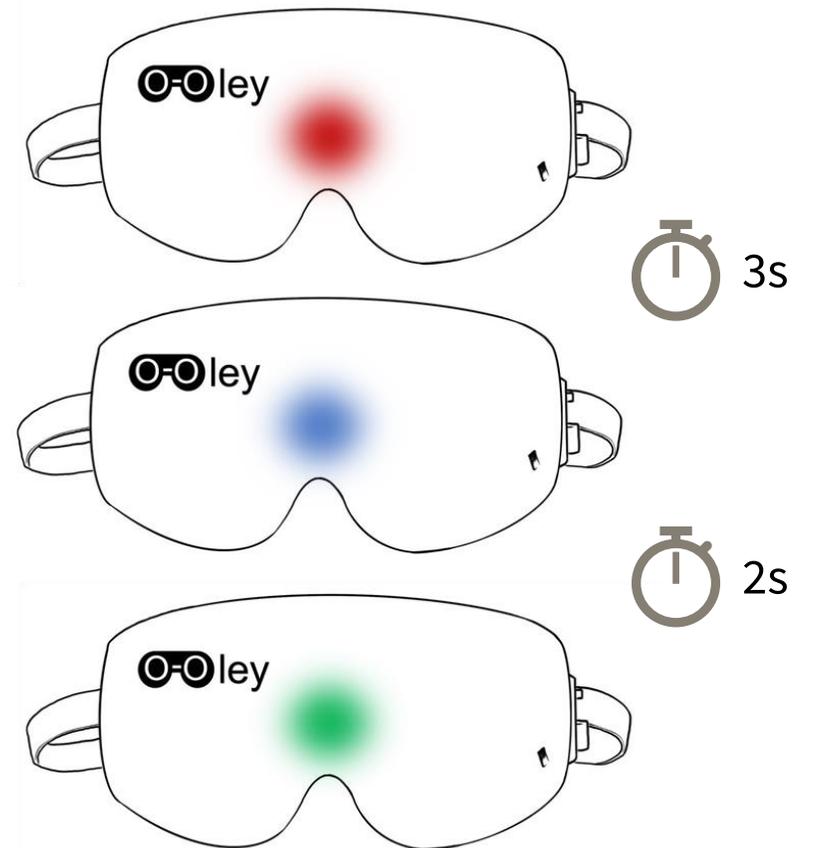

Getting Started

4. Place the device on clean, dry skin and over the eyes. Ensure that eyeglasses and contact lenses are removed before use.

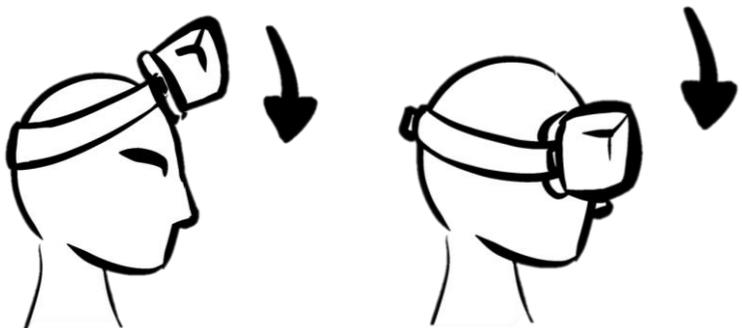

5. Adjust the strap over your head and tighten it.



6. Enjoy the therapeutic effects on the eyes.



7. Once you are done with your eye therapy session, turn off the device by pressing the white button once followed by the power button twice.

Charging

- O-Oley is USB-C enabled
- The USB port is located at the bottom side of the device near the power button
- Insert the USB-C cable into the port to charge O-Oley
- The device is fully charged when the lights are no longer blinking

Precautions

Warning

- O-Oley is not waterproof and not water-resistant
- Do not open the case/shell of the device

Cleaning the goggle

- Wipe with clean soft dry cloth
- Use a diluted neutral detergent to clean



If I care more, eyes stress less

